

RESTORATION

DENITAI
CATALOG

INNOVATIONS FOR RESTORATIVE DENTISTRY

For over 30 years, COLTENE has been a renowned provider of high grade restorative products, which have ever since developed into a focus product segment. Here the emphasis is on innovative and clinically proven products for aesthetic filling therapy, as well as a wide range of luting materials.

COLTENE pioneered the development of white tooth fillings to replace amalgam filling materials which were the standard in early days. As early as 1983, the micro-hybrid composite BRILLIANT was introduced, which is now an integral part of restorative dentistry.

Collaboration with leading universities and opinion leaders is an important success factor, which has repeatedly led to groundbreaking new developments. This is how the BRILLIANT-Esthetic Line filling concept enabled intraoral as well as extraoral restoration thanks to a special polymerisation furnace, which was far ahead of its time. To simplify shade selection, a "Duo Shade" system was developed for the SYNERGY composite. Thanks to the excellent self-blend properties, the entire shade spectrum is covered with just a few shades.

With MIRIS, the premium filling material from COLTENE, a completely new shade concept "Natural Layering" was implemented in collaboration with Dr. Didier Dietschi, University of Geneva. Here the dentine and enamel layers of the natural tooth are reproduced one to one in order to satisfy even the most demanding aesthetic requirements.

With the development of the first Direct Composite Veneering System, COMPONEER, composite enamel shells were presented for direct restoration of anterior teeth for the first time. Your patients can be provided with a natural aesthetic smile – in just one sitting! An attractive and successful treatment alternative for the patient and the dentist. In addition, a broad spectrum of outstanding adhesive systems, dependable luting cements, and also as core build-up materials, polymerisation lamps and much more besides round off the product portfolio and portray the company's innovative approach.



Company

COLTENE is a global leader in the development, manufacture and sale of consumables and small-size equipment for dental treatment applications.


The Dental Group offers a wide range of products, encompassing six segments – solutions for almost all dental treatments. The complete endodontology programme includes root canal preparation with files and rinsing solutions, obturation and post systems. The next extensive range is centred entirely on restorative filling therapy. This particularly includes temporary and permanent filling materials, as well as adhesive systems and polymerisation lamps. Prosthetics represents a further main area. It includes a wide spectrum of condensation and addition cross-linking impression materials. The segments of treatment aids, infection control and rotating instruments round off the programme.

COLTENE has time and time again developed pioneering innovations for over 50 years, and Roeko for over a 100 years who offer the user first grade dental solutions.

Today the COLTENE Group operates production facilities in Switzerland, Germany, Brazil and the USA. It maintains a worldwide sales network with its own subsidiaries and distribution partners in over 120 countries. COLTENE therefore serves dentists, dental technicians, opinion leaders and universities with tailored quality products and services – worldwide.

www.coltene.com

History

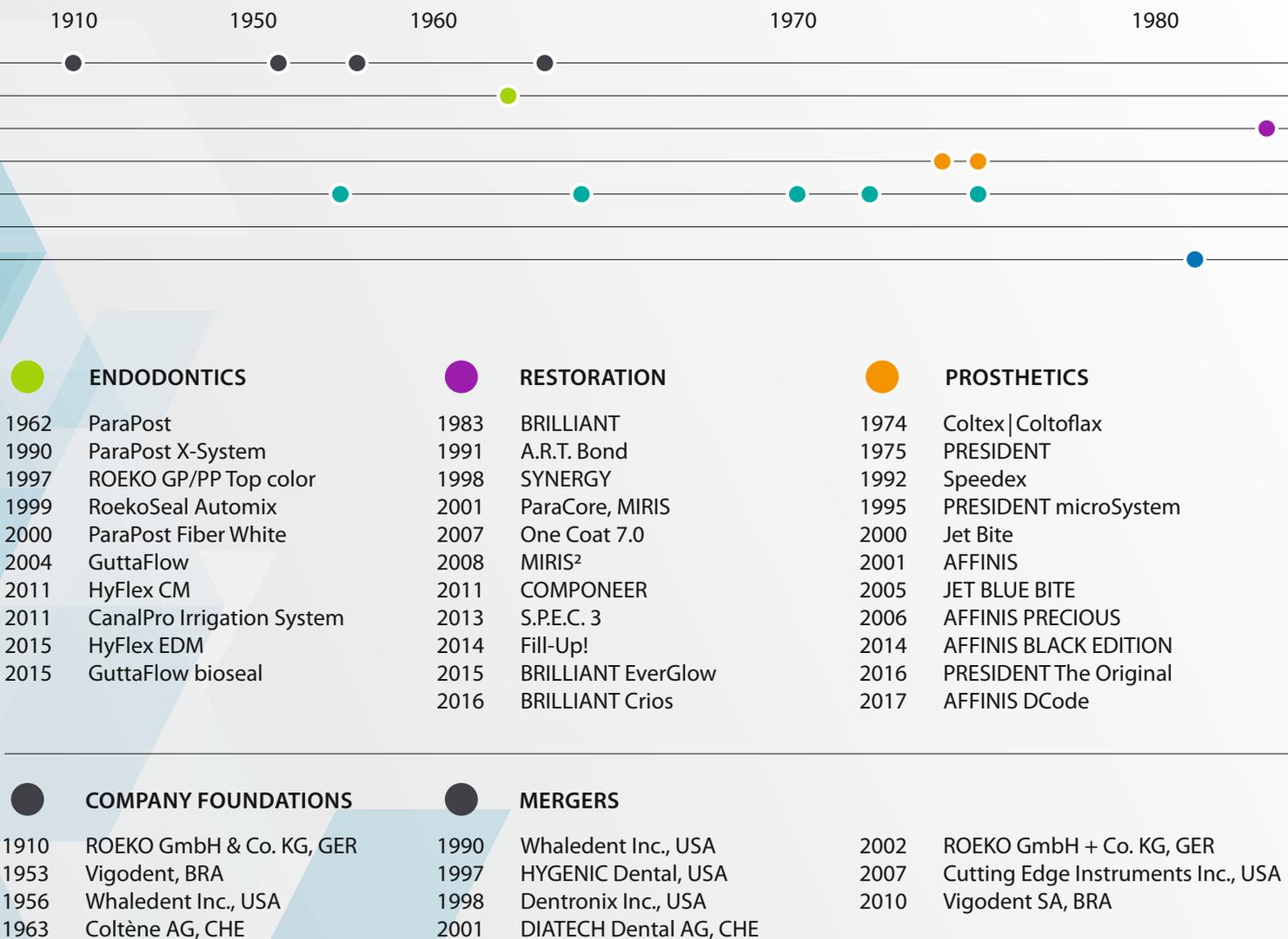

Production facilities

Switzerland: Coltène/Whaledent AG, Headquarter in Altstätten. Development and production of chemical dental consumables and rotary instruments.


Germany: Coltène/Whaledent GmbH + Co. KG in Langenau. Development and production of treatment auxiliaries and endodontic products.

USA: Coltène/Whaledent Inc. in Cuyahoga Falls. Production of mechanical-technical dental products and orthodontics.

Brazil: Vigodent SA Indústria e Comércio in Rio de Janeiro. Production of chemical dental consumables.



COLTENE Headquarter, Altstätten / Switzerland

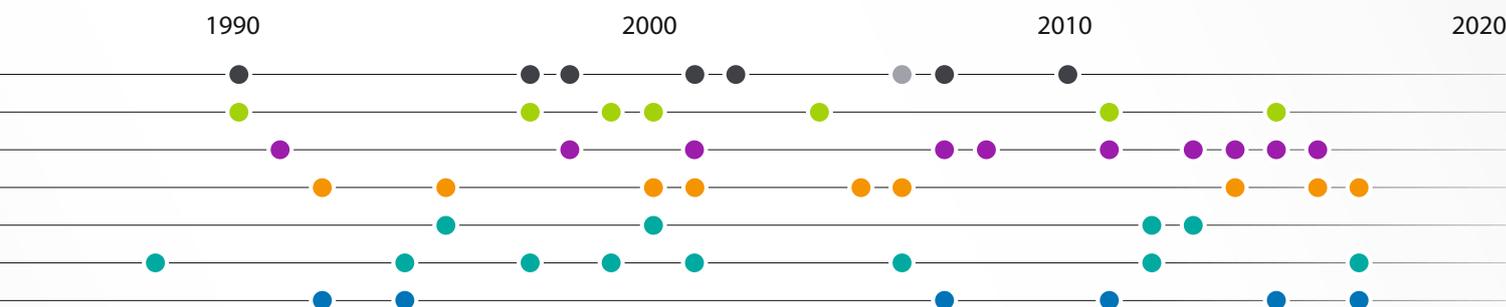

TREATMENT AUXILIARIES

- 1955 Luna
- 1964 Parotissroll
- 1970 Cotton Pellets
- 1972 Surgitip
- 1975 Gelatamp
- 1995 Dental Dam Silicone non latex
- 2000 Flexi Dam non latex
- 2012 Elasti-Dam
- 2013 Surgitip-endo

INFECTION CONTROL

- 1988 BioSonic UC1 Cleaner
- 1994 Steri-Quick
- 1997 Protecta Quick non latex
- 1999 Simplex-Plus Maxi
- 2001 BioSonic US100 Scaler
- 2006 BioSonic UC125
- 2012 Protecta active carbon
- 2017 BioSonic UC150

ROTARY INSTRUMENTS

- 1981 Diamonds
- 1992 Carbides
- 1994 Topspin
- 2007 Speedster
- 2011 SwissFlex
- 2015 ShapeGuard
- 2017 Z-Rex

INITIAL PUBLIC OFFERING

- 2006 IPO of COLTENE Holding AG at SIX Swiss Exchange, CHE

**INDICATION
PRODUCT TYPE**

**FIELD OF APPLICATION
PROPERTIES**

PRODUCT NAME

ETCHING

Etching gel	35% phosphoric acid	Etchant Gel S	10
--------------------	---------------------	---------------	-----------

BONDING

Light-curing bond	One-component enamel adhesive	Margin Bond	10
	Classical dentine enamel bond	A.R.T. Bond	10
	Self-Etching Bond	One Coat Self-Etching Bond	10
	Total-Etch dentine enamel bond	One Coat Bond	11
	Self-Etching one component adhesive	One Coat 7.0	11
	Universal Bond	ONE COAT 7 UNIVERSAL	11
Chemically curing bond	Self-conditioning adhesive system	ParaBond	11

FILLING MATERIALS

Light-curing composites	Aesthetic filling material	MIRIS ²	15
	Universal submicron hybrid	BRILLIANT EverGlow	17
	Submicron hybrid flow material	BRILLIANT EverGlow Flow	18
	Universal nano composite	SYNERGY D6	19
	Nanofilled flow material	SYNERGY D6 Flow	20
	Nanohybrid composite	SYNERGY Nano Formula	21
Dual-curing composite	Flowable bulk composite	Fill-Up!	23

Accessories

Matrix bands	Auxiliary material	Dispenser for composite tips	24
		Mixing pad	24
Auxiliary material	Composite-based effect shades	Paint On Colors	24
		For amalgam fillings and glass ionomer cements	ROEKO matrix bands
Effect shades	For approximal separation	ROEKO steel separating strips	24
Grinding and polishing	For smoothing the interproximal space	ROEKO abrasive and polishing strips	25
	For interproximal contouring, finishing and high luster polishing of filling materials	DIATECH SwissFlex Kit	25
	Polishing Kits for composite restorations	DIATECH Finishing & Polishing Kits for Composite	25
	High performance polishers for CAD/CAM composite blocs	DIATECH Finishing & Polishing Kits for BRILLIANT Crios (chairside/lab)	26

**INDICATION
PRODUCT TYPE**

**FIELD OF APPLICATION
PROPERTIES**

PRODUCT NAME

CURING LIGHTS

Light polymerisation	High performance curing light	S.P.E.C. 3	27
	Standard LED	Coltolux LED	27

TEMPORARY RESTORATION

Crowns & Bridges	For the fabrication of temporary crowns and bridges	Cool Temp NATURAL	28
	A-silicone for temporary cementation	TempoSIL 2	29
Cavities	Dual-curing temporary filling material	DuoTEMP	28
	Chemically curing filling material	Coltosol F	29
Accessories	For easy removal of temporaries	Intraoral Separator	29

COMPOSITE ENAMEL SHELLS

Front tooth veneers	Nano-hybrid composite enamel shells	COMPONEER	34
Accessories	For modelling composites and placing COMPONEER	Modelling instrument MBC and more	37
	For preparing and finishing of COMPONEER	DIATECH COMPONEER Preparation and Finishing Kit	37

CAD/CAM SOLUTIONS

Blocs	Permanent composite restoration	BRILLIANT Crios	39
--------------	---------------------------------	-----------------	----

CORE BUILD-UP AND CEMENTS

Core build-up material	Dual curing material for core build-up and cement	ParaCore	42
Cements	Dual curing cement in the automix syringe	DuoCem	43
	Self-adhesive, dual curing composite-based cement	SoloCem	45

PARAPULPAL PINS

Pins	Self-threading parapulpal retention pins	MAX Pin	46
		TMS LINK/ TMS LINK Plus	46
		TMS	49
	Colour-coded drills for the TMS family	KODEX drills	48

Etching
& Bonding

Restoration

LIGHT CURING ADHESIVE SYSTEMS

SELECTIVE-ETCH


Margin Bond

1st generation
Enamel is etched



A.R.T. Bond


3rd generation
Enamel is etched
Dentine is self-conditioned

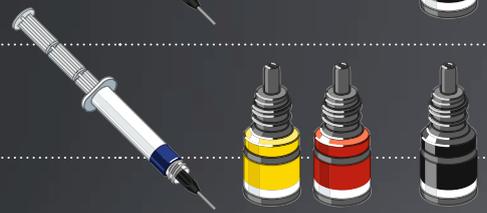

TOTAL-ETCH

One Coat Bond


5th generation
Enamel is etched
Dentine is etched

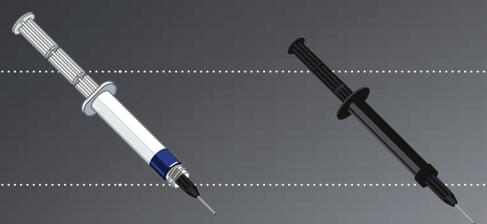

SELF-ETCH

One Coat Self-Etching Bond

6th generation
Enamel is self-conditioned
Dentine is self-conditioned

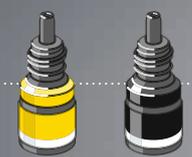

One Coat 7.0

7th generation
Enamel is self-conditioned
Dentine is self-conditioned



UNIVERSAL

ONE COAT 7 UNIVERSAL


Enamel is self-conditioned
Dentine is self-conditioned



CHEMICALLY-CURING ADHESIVE SYSTEMS


ParaBond

Enamel is etched with Etchant Gel or
Non-Rinse Conditioner
Dentine is self-conditioned

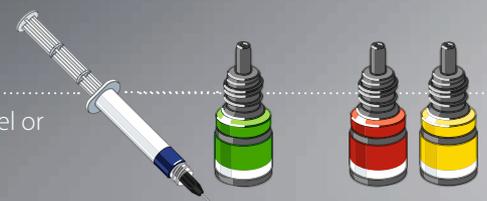

One Coat 7.0 Activator

Enamel is self-conditioned
Dentine is self-conditioned

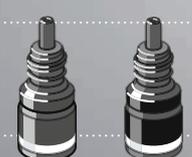

Etchant Gel S

35 % phosphoric acid etching gel with precise placement for use on enamel and dentine.



Etchant Gel S Kit	7373
Syringes	2 × 2.5 ml
Application Needles	20 pcs

Etchant Gel S Refill	35 %	5 ml	7374
Application Needles	black	20 pcs	7876

Margin Bond

Light curing one component enamel adhesive for adhesive tooth restoration.



Margin Bond	7367
Light-cured enamel bond	5 ml

A.R.T. Bond

Classical light curing dentine/enamel bonding system for universal indications, high bond strength, withstands the test of time, scientifically recommended.



A.R.T. Bond Kit	7362
Primer A	5 ml
Primer B	5 ml
Bond	5 ml
Accessories	

Bond Refill		5 ml	7363
Primer A & B Refill		2 × 5 ml	7364

One Coat Self-Etching Bond

Formulated to provide an easy technique, One Coat Self-Etching Bond eliminates the need for an extra etching step. Primer 1 solubilizes the smear layer, without totally removing it as total etch bonds do. Bond 2 penetrates through the smear layer into the tubules to create the so called resin tags, which are responsible for its outstanding retention.



One Coat Self-Etching Bond Introkit	7630
Primer 1	5 ml
Bond 2	5 ml
Brushes yellow	50 pcs
Brushes black	50 pcs
Mixing well	1 pc

One Coat Self-Etching Primer 1 Refill		5 ml	7635
One Coat Self-Etching Bond 2 Refill		5 ml	7636

Brushes yellow		50 pcs	7633
Brushes black		50 pcs	7634

One Coat Bond



Total-Etch light curing dentin/enamel bonding system for universal indications, easy application due to non-drip gel consistency, solvent-free, prevents postoperative sensitivity.

One Coat Bond Intro Kit	60011757
One Coat Bond	1.2 ml
Etchant Gel S (35 % phosphoric acid)	2.5 ml
Brushes black	50 pcs
Application Needles	20 pcs

One Coat Bond Refill	1.2 ml	7879
One Coat Bond Refill	2 × 1.2 ml	7873
Brushes black	50 pcs	7634
Application Needles	black	20 pcs 7876

One Coat 7.0



7th generation, single component, light curing, self-etching adhesive designed to etch, prime and bond all in one solution for "effortless bonding performance" on any surface!

One Coat 7.0 Intro Kit	7047
One Coat 7.0	5 ml
Brushes black	50 pcs
Mixing well	1 pc
Cover, mixing well	1 pc

One Coat 7.0 Activator Kit	7054	
One Coat 7.0 Activator	3 ml	
Brushes black	50 pcs	
One Coat 7.0 Refill	5 ml	7049
Brushes black	50 pcs	7634

ParaBond



Chemically cured, self-conditioning adhesive system for enamel and dentine which consists of a non-rinse conditioner and a chemically curing adhesive (adhesives A and B).

ParaBond Adhesive	7486
Chemical-cured bond	
ParaBond Non-Rinse Conditioner	3 ml
ParaBond Adhesive A	3 ml
ParaBond Adhesive B	3 ml

ParaBond Adhesive A & B Refill	7493	
ParaBond Adhesive A	3 ml	
ParaBond Adhesive B	3 ml	
ParaBond Non-Rinse Conditioner Refill	3 ml	7494

ONE COAT 7 UNIVERSAL



Light-cured, one component bonding agent used with the self-etch, selective etch or total etch technique for adhesive restoration techniques.

ONE COAT 7 UNIVERSAL Intro Kit	60019538
ONE COAT 7 UNIVERSAL	5 ml
Etchant Gel S	2.5 ml
Brushes black	50 pcs
Mixing well	1 pc
Cover, mixing well	1 pc

One Coat 7.0 Activator Kit	7054
One Coat 7.0 Activator	3 ml
Brushes black	50 pcs

ONE COAT 7 UNIVERSAL Single Dose	60019540
ONE COAT 7 UNIVERSAL Single Dose	50 × 0.1 ml
Brushes black	50 pcs

ONE COAT 7 UNIVERSAL Refill	5 ml	60019539
Etchant Gel S Refill	5 ml	7374
Application Needles	20 pcs	7876
Brushes black	50 pcs	7634

Restoration

Filling Materials

MIRIS®2

Highly aesthetic composite system based on the innovative Natural Layering Technique, matched to young, adult or older patients. The Shade Guide allows the dentist intuitive shade selection and guarantees unrivalled aesthetics.

8 dentine, 5 enamel and 4 effect shades



SYNERGY® D6

Universal composite using the intelligent Duo Shade System which applies for 6 of the dentine shades. The optical self-blend properties of SYNERGY D6 allow a broad indication spectrum to be covered with just a few shades.

8 dentine and 2 enamel shades.
Duo Shade / VITA™-based



BRILLIANT EverGlow®

The latest, most advanced universal composite of COLTENE. Developed to provide an ideal mixture of long-lasting aesthetics, handling convenience and mechanical strength, it is a true all-round material.

3 opaque, 7 universal, 2 translucent shades.
Duo Shade / VITA™ based




Fill-Up!®

Medium viscous, dual curing bulk filling composite for posterior Class I and II restorations. Owing to its dual curing property, fillings can be administered in arbitrary depth, without the need of an additional covering layer. Fill-Up! is the material of choice whenever a fast restoration is needed or for cavities difficult to reach with the curing light.

1 universal shade




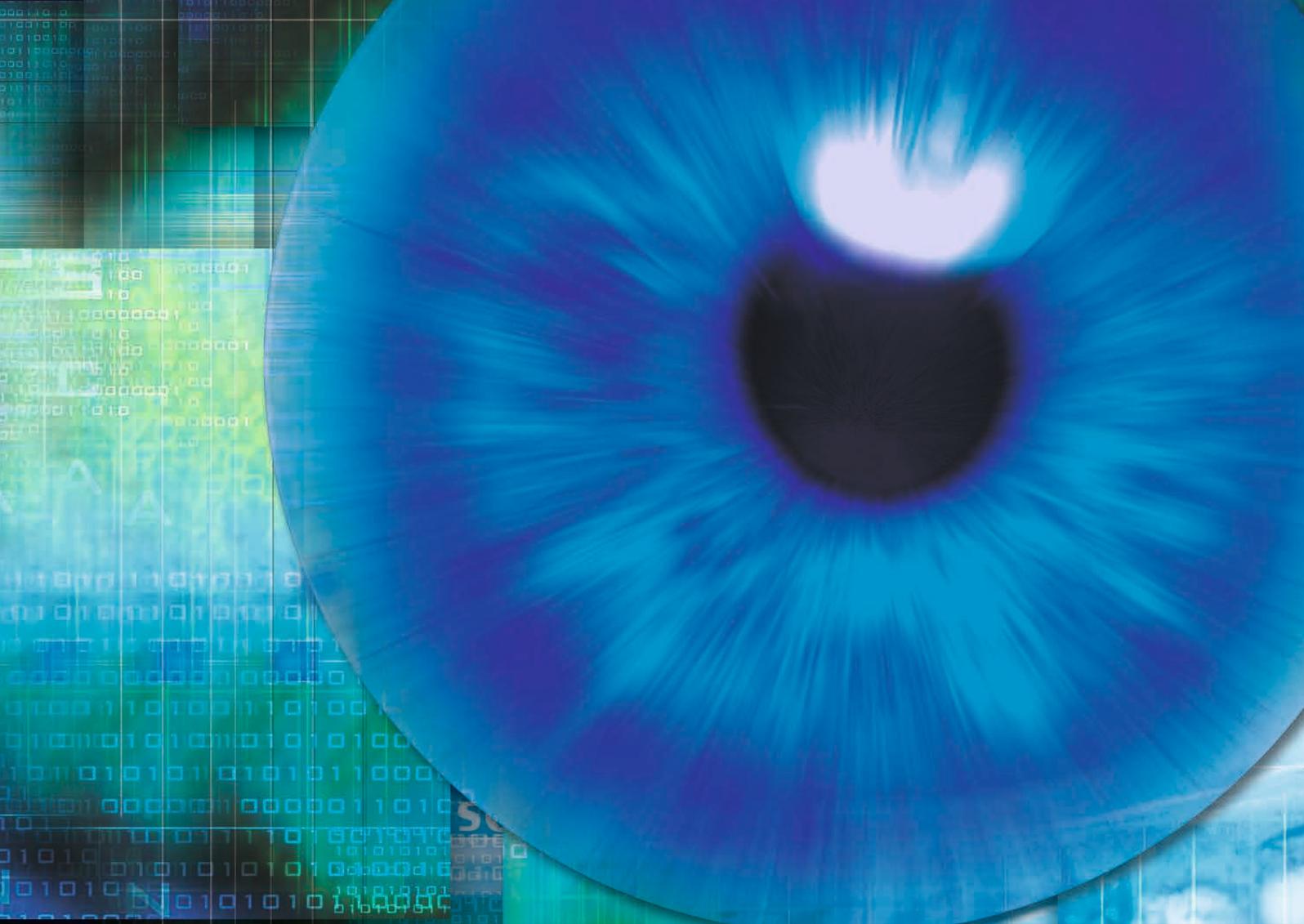

Miris^{®2}

Artificial reality

MIRIS²

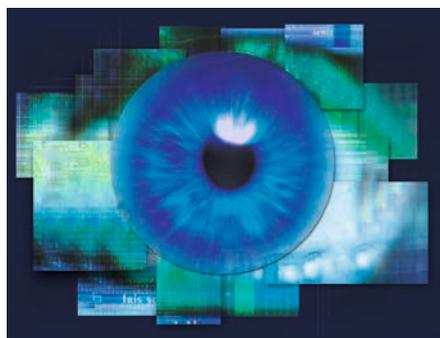


Artificial reality

MIRIS² takes you to another level of the ideal creation of 'artificial reality'. The unique shade system of MIRIS² is based on the natural layering concept and works with dentin and enamel masses whose optical properties are comparable with those of the original tissue. For this purpose, the shades and translucency of natural teeth in adolescent, adult and elderly patients were measured and emulated. The composite-based MIRIS² shade guide is separated into dentin core and enamel shells that can be freely interposed with one another to achieve an aesthetically optimal definition of shade.

In addition to dentin and enamel shades, special effect shades complete the system to allow customised reconstruction, taking special aspects into account, and thus achieve unsurpassed aesthetics – even in highly complex cases.

MIRIS² Kits

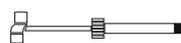


MIRIS ² Syringe Set	8431	MIRIS ² Tips Set	8465
13 Syringes (4 g / 2 ml each), 4 Syringes (2.3 g / 1.4 ml each), Accessories		78 Tips (0.25 g/0.125 ml each), 4 Syringes (2.3 g / 1.4 ml each), Accessories	
MIRIS ² Dentin Shade	8 pcs	MIRIS ² Dentin Shade	6 pcs each
Shade 0 (S0), Shade 1 (S1), Shade 2 (S2), Shade 3 (S3), Shade 4 (S4), Shade 5 (S5), Shade 6 (S6), Shade 7 (S7)		Shade 0 (S0), Shade 1 (S1), Shade 2 (S2), Shade 3 (S3), Shade 4 (S4), Shade 5 (S5), Shade 6 (S6), Shade 7 (S7)	
MIRIS ² Enamel Shades	5 pcs	MIRIS ² Enamel Shades	6 pcs each
White Regular (WR), White Bleach (WB), Neutral Regular (NR), Neutral Trans (NT), Ivory Regular (IR)		White Regular (WR), White Bleach (WB), Neutral Regular (NR), Neutral Trans (NT), Ivory Regular (IR)	
MIRIS ² Effect Shade	4 pcs	MIRIS ² Effect Shade	4 pcs
Blue (B), White (W), White Opaque (WO), Gold (G)		Blue (B), White (W), White Opaque (WO), Gold (G)	
Shade Guide		Shade Guide	
Accessories		Accessories	

Material selection according to the clinical situation

Teeth	Bleached			Adolescents			Adults			Elderly patients		
	Dentin	Effect	Enamel	Dentin	Effect	Enamel	Dentin	Effect	Enamel	Dentin	Effect	Enamel
Classes I-II	S2	-	WB	S2	-	WR	S3	-	WR/NR	S4	-	NR
Class III	S0-S1	-	WB	S1-S2	-	WR/WB	S2-S3	-	WR/NR/IR	S4-S5	-	NR/IR/NT
Class IV	S0-S1	B/W/WO	WB	S1-S2	B/W/WO	WR/WB	S2-S3	B/G/W/WO	WR/NR/IR	S4-S5	G/W/WO	NR/IR/NT
Class V	S0-S3	-	WB	S1-S3	-	WR/WB	S2-S5	-	WR/NR/IR	S4-S7	-	NR/IR/NT
Aesthetics	S0-S1	B/W/WO	WB	S1-S2	B/W/WO	WR/WB	S2-S3	B/G/W/WO	WR/NR/IR	S4-S5	B/G/W/WO	NR/IR/NT
Veneers	(S0-S1)	B/W/WO	WB	(S1-S2)	B/W/WO	WR/WB	S2-S3	B/G/W/WO	WR/NR/IR	S4-S5	B/G/W/WO	NR/IR/NT

The effect shades Gold, White and White Opaque can be mixed so it is possible to produce any desired opaque tint in all dentine shades (S0 to S7).

MIRIS² Refills

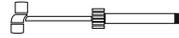

Dentin Shade – Syringe		4 g
Shade 0 (S0)	1 pc	8409
Shade 1 (S1)	1 pc	8432
Shade 2 (S2)	1 pc	8433
Shade 3 (S3)	1 pc	8434
Shade 4 (S4)	1 pc	8435
Shade 5 (S5)	1 pc	8436
Shade 6 (S6)	1 pc	8437
Shade 7 (S7)	1 pc	8438

Dentin Shade – Tips (0.25 g)			10 pcs	20 pcs
Shade 0 (S0)			8406	8408
Shade 1 (S1)			8472	8486
Shade 2 (S2)			8473	8487
Shade 3 (S3)			8474	8488
Shade 4 (S4)			8475	8489
Shade 5 (S5)			8476	8490
Shade 6 (S6)			8477	8491
Shade 7 (S7)			8478	8492

Enamel Shade – Syringe		4 g
White Regular (WR)	1 pc	8439
White Bleach (WB)	1 pc	8440
Neutral Regular (NR)	1 pc	8441
Neutral Trans (NT)	1 pc	8442
Ivory Regular (IR)	1 pc	8443

Enamel Shade – Tips (0.25 g)			10 pcs	20 pcs
White Regular (WR)			8479	8493
White Bleach (WB)			8480	8494
Neutral Regular (NR)			8481	8495
Neutral Trans (NT)			8482	8496
Ivory Regular (IR)			8483	8497

Effect Shade – Syringe		2.3 g
White (W)	1 pc	8445
White Opaque (WO)	1 pc	8446
Gold (G)	1 pc	8447
Blue (B)	1 pc	8403

Accessories		
MIRIS ² Shade Guide		8411
Application Needles	10 pcs	7874
Composite Tip Dispenser		6671

BRILLIANT EverGlow



Long-lasting brilliance

BRILLIANT EverGlow is a universal composite of the latest generation and a true all-round material. It has been developed with special focus on long-lasting aesthetics and handling convenience. The shade assortment comprises shades in three opacities: 3 opaque masking shades, 7 universal Duo Shades suitable for single-shade restorations and 2 translucent shades for optional enamel imitation. Meeting highest requirements for “brilliant” anterior and posterior restorations, it is the ideal modern filling material for everyday dentistry:

- Exceptional polishability
- Excellent gloss retention
- Aesthetic single-shade restorations
- Smooth consistency
- Good wettability, low stickiness to the instrument
- High abrasion resistance
- High compressive strength
- High resistance to operating lights
- VITA™ based Duo Shade system

* Vita™ is not a trademark of Coltène/Whaledent



	Opaque Shades			Universal Shades							Translucent Shades	
	Opaque Bleach (OBL)	Opaque A1 (OA1)	Opaque A3 (OA3)	Bleach (BL)	A1 / B1	A2 / B2	A3 / D3	A3.5 / B3	C2 / C3	A4 / C4	Bleach Translucent (BL Trans)	Translucent (Trans)
Trial Kits												
60019720 4 Syringes à 3g						1	1	1				1
60019721 32 Tips à 0.2g						8	8	8				8
Anterior Kits												
60019724 4 Syringes à 3g				1	1						1	1
60019725 32 Tips à 0.2g				8	8						8	8
System Kits												
60019722 9 Syringes à 3g				1	1	1	1	1	1	1	1	1
60019723 60 Tips à 0.2g				5	8	8	8	8	5	5	5	8
Regular Kits												
60019726 9 Syringes à 3g		1	1		1	1	2	1	1			1
60019727 60 Tips à 0.2g		5	5		5	10	12	10	5			8

Included accessories: Instruction for use, mixing pad, Shade Guide

BRILLIANT EverGlow®



BRILLIANT EverGlow Refills Syringes		3 g
Opaque Bleach (OBL)	1 pc	60019743
Opaque A1 (OA1)	1 pc	60019744
Opaque A3 (OA3)	1 pc	60019745
Bleach (BL)	1 pc	60019697
A1 / B1	1 pc	60019698
A2 / B2	1 pc	60019699
A3 / D3	1 pc	60019700
A3.5 / B3	1 pc	60019701
C2 / C3	1 pc	60019702
A4 / C4	1 pc	60019703
Translucent (Trans)	1 pc	60019704
Bleach Translucent (BL Trans)	1 pc	60019705

BRILLIANT EverGlow Refills Tips		0,2 g
Opaque Bleach (OBL)	20 pcs	60019746
Opaque A1 (OA1)	20 pcs	60019747
Opaque A3 (OA3)	20 pcs	60019748
Bleach (BL)	20 pcs	60019706
A1 / B1	20 pcs	60019707
A2 / B2	20 pcs	60019708
A3 / D3	20 pcs	60019709
A3.5 / B3	20 pcs	60019710
C2 / C3	20 pcs	60019716
A4 / C4	20 pcs	60019717
Translucent (Trans)	20 pcs	60019718
Bleach Translucent (BL Trans)	20 pcs	60019719

BRILLIANT EverGlow Flow

BRILLIANT EverGlow Flow is the perfect complement to the universal composite BRILLIANT EverGlow. The material provides a low viscous consistency but good stability and thixotropy, allowing a controlled and comfortable application.

BRILLIANT EverGlow Flow Dentin Kit								
	Opaque A2 (OA2)	Bleach (BL)	A1 / B1	A2 / B2	A3 / D3	A3.5 / B3	A4 / C4	Translucent (Trans)
60019760 4 Syringes à 2g			1	1	1	1		

Included accessories: Instruction for use, mixing pad, 40 application needles



BRILLIANT EverGlow Flow Refills (1 × 2 g + Application needles)		
Opaque A2 (OA2)	1 pc	60019751
Bleach (BL)	1 pc	60019752
A1 / B1	1 pc	60019753
A2 / B2	1 pc	60019754
A3 / D3	1 pc	60019755
A3.5 / B3	1 pc	60019756
A4 / C4	1 pc	60019757
Translucent (Trans)	1 pc	60019758
Flow Application Needles, Standard, Ø 0.8 mm	20 pcs	8469
Flow Application Needles, Fine, Ø 0.4 mm	20 pcs	60022137

SYNERGY D6



Nano composite for everyday dentistry.

SYNERGY D6 is a nano-hybrid composite for everyday dentistry. A special feature is the Duo Shade system which applies for 6 of the dentin shades. Thanks to the excellent optical self-blending properties one duo shade matches 2 VITA™ shades (e.g. A1/B1 or A2/B2). As a result a small number of shades is sufficient to cover a wide range of tooth shades. In total, the assortment of SYNERGY D6 offers 8 dentin shades and 2 enamel shades.




Restoration using SYNERGY D6
Dentin Shade A2/B2

Restoration using SYNERGY D6
Dentin Shade A2/B2 and Enamel Universal

* Vita™ is not a trademark of Coltène/Whaledent

SYNERGY D6 Intro Kits

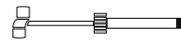

SYNERGY D6 Intro Kit Syringes 8362

8 Syringes (4 g/2 ml each) and Accessories

SYNERGY D6 Dentin Shades:

White Bleach, A1/B1, A2/B2, A3/D3, A3.5/B3, C2/C3

SYNERGY D6 Enamel Shades:

Universal, White Opalescent

Shade Guide

Accessories

SYNERGY D6 Intro Kit Tips 8363

80 Tips (0.25 g/0.125 ml each) and Accessories

SYNERGY D6 Dentin Shades:

White Bleach, A1/B1, A2/B2, A3/D3, A3.5/B3, C2/C3

SYNERGY D6 Enamel Shades:

Universal, White Opalescent

Shade Guide

Accessories

SYNERGY D6 Refills



SYNERGY D6 Syringes Refills		4 g/2 ml
Dentin Bleach Opaque (BL/O)	1 pc	60013814
Dentin White Bleach (WB)	1 pc	8365
Dentin A1/B1	1 pc	8366
Dentin A2/B2	1 pc	8367
Dentin A3/D3	1 pc	8368
Dentin A3.5/B3	1 pc	8369
Dentin C2/C3	1 pc	8370
Dentin A4/C4	1 pc	8384
Enamel Universal	1 pc	8371
Enamel White Opalescent (WO)	1 pc	8372

SYNERGY D6 Accessories		
Shade Guide	1 pc	8410
Composite Dispenser	1 pc	6671

SYNERGY D6 Tips Refills		0.25 g/0.125 ml
Dentin Bleach Opaque (BL/O)	1 pc	60013813
Dentin White Bleach (WB)	10 pcs	8375
Dentin A1/B1	10 pcs	8376
Dentin A2/B2	10 pcs	8377
Dentin A3/D3	10 pcs	8378
Dentin A3.5/B3	10 pcs	8379
Dentin C2/C3	10 pcs	8380
Dentin A4/C4	10 pcs	8383
Enamel Universal	10 pcs	8381
Enamel White Opalescent (WO)	10 pcs	8382

SYNERGY D6 Flow

SYNERGY D6 Flow is the perfect complement to the universal composite SYNERGY D6. The formula of Synergy D6 Flow is based on the technology of SYNERGY D6. The material is stable, yet flows gently under pressure to ensure easy handling and precision application.



SYNERGY D6 Flow Intro Kit		7650
A2/B2, A3/D3, White Bleach (WB), Enamel Universal	4 x 2.3 g	
Bent Application Needles	32 pcs	

SYNERGY D6 Flow Refills		2.3 g
A1/B1	1 pc	7651
A2/B2	1 pc	7652
A3/D3	1 pc	7653
A3.5/B3	1 pc	7654
A4/C4	1 pc	7655
White Bleach (WB)	1 pc	7656
Enamel Universal	1 pc	7657

SYNERGY D6 Flow Dentin Intro Kit		7660
SYNERGY D6 Flow A1/B1, A2/B2, A3/D3, A3.5/B3	4 x 2.3 g	
Bent Application Needles	32 pcs	

SYNERGY D6 Accessories		
Shade Guide SYNERGY D6	1 pc	8410
Flow Application Needles Standard, Ø 0.8 mm	20 pcs	8469

SYNERGY Nano Formula

SYNERGY Nano Formula is a radiopaque, highly-filled, nano-hybrid composite. It is available in two consistencies named SYNERGY Nano Formula Duo Shade and SYNERGY Nano Formula Compact.

SYNERGY Nano Formula Duo Shade

SYNERGY Nano Formula Duo Shade is a smooth, non-sticky universal composite, covering a wide spectrum of tooth shades thanks to the duo shade properties.




Duo Shade Syringes Refills		4 g
A1/D2	1 pc	7821
A2/B2	1 pc	7822
A3/D3	1 pc	7823
A3.5/B3	1 pc	7824
A4/M5	1 pc	7825
C2/C3	1 pc	7826



Duo Shade Tips Refills		0.25 g
A1/D2	20 pcs	7831
A2/B2	20 pcs	7832
A3/D3	20 pcs	7833
A3.5/B3	20 pcs	7834
A4/M5	20 pcs	7835
C2/C3	20 pcs	7836

SYNERGY Nano Formula Compact

SYNERGY Nano Formula Compact is a packable composite, offering an alternative to amalgam for large restorations.

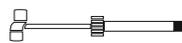

Compact Syringes Refills		4 g
A2/B2	1 pc	7491
A3/D3	1 pc	7492



Compact Tips Refills		0.25 g
A2/B2	20 pcs	7837
A3/D3	20 pcs	7838
A3.5/B3	20 pcs	7839

Hole in One

Filling in a single step with **Fill-Up!®**



Fill-Up!® - in a single step to give a perfect result

- optimal depth polymerisation with minimal shrinkage due to dual curing system restoration
- guaranteed single-layer technique - even in very deep cavities of 10 mm
- optimised sealing of margins - reduced postoperative sensitivity
- universal shade in a convenient automix syringe for efficient placement

Deep. Fast. Perfect.

www.coltene.com




COLTENE

Fill-Up!

Fill-Up! is a radiopaque, medium viscous, dual curing composite material of the newest generation used for bulk filling therapy in the posterior dental region. It combines the advantages of resin-based composites with a simplified and time-efficient handling. Owing to its dual curing property, fillings with Fill-Up! can be administered in arbitrary depth, without the need of an additional covering layer. Fill-Up! is the material of choice for all Class I and II fillings, cavity linings and core build-ups – every time when a fast restoration is needed or for cavities difficult to reach with the curing light.

- Arbitrary filling depth
- Minimised shrinkage stress
- Optimised marginal seal
- One layer
- Universal colour, fluorescent and radiopaque
- Contains zinc oxide
- Fast and easy procedure

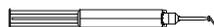

Fill-Up! Intro Kit with ParaBond	60019342
Fill-Up! Universal	2 × 4.5 g
Etchant Gel S	1 × 2.5 ml
ParaBond Adhesive A & B	2 × 3 ml
Mixing Tip Short Super Fine	6 pcs
Mixing Tip Short Fine	8 pcs

Fill-Up! Intro Kit with ONE COAT 7 UNIVERSAL	60019973
Fill-Up! Universal	2 × 4.5 g
ONE COAT 7 UNIVERSAL	1 × 5 ml
One Coat 7.0 Activator	1 × 3 ml
Mixing Tip Short Super Fine	6 pcs
Mixing Tip Short Fine	8 pcs

Fill-Up! Economy Kit	60019355
Fill-Up! Universal	5 × 4.5 g
Mixing Tip Short Super Fine	15 pcs
Mixing Tip Short Fine	20 pcs

Fill-Up! Refill	60019341
Fill-Up! Universal	1 × 4.5 g
Mixing Tip Short Super Fine	3 pcs
Mixing Tip Short Fine	4 pcs

Mixing Tip Short Super Fine	6747
Mixing Tip, Ø 1 mm	40 pcs

Mixing Tip Short Fine	6759
Mixing Tip, Ø 1.8 mm	40 pcs

Etchant Gel S Kit	7373
Syringes	2 × 2.5 ml
Application Needles	20 pcs

ParaBond Adhesive A & B Refill	7493
ParaBond Adhesive A	3 ml
ParaBond Adhesive B	3 ml

ONE COAT 7 UNIVERSAL Refill	60019539
ONE COAT 7 UNIVERSAL	1 × 5 ml

Auxiliary material

Auxiliary material		
Dispenser for Composite Tips	1 pc	6671
Mixing pad	1 pc	7080

Paint On Color



Paint on Color are effect colours for an individual characterisation of a composite restoration. The colours can be mixed to individual additional colours (orange = red+yellow, purple = red+blue, green = blue+yellow) or brightened up by adding white.

Paint On Color Kit	7330
White, Red, Grey, Yellow, Brown, Blue	6 x 3 ml
White Opaque	2 x 3 ml

Paint On Color Refills	1 x 3 ml
White	7332
Grey	7333
Blue	7334
Red	7335
Yellow	7336
Brown	7337
White Opaque	7338

ROEKO Matrix Bands



For use with amalgam fillings and glass ionomer cement.

ROEKO Steel Matrix Band soft		
Stainless steel, extra fine (0.05 mm), can be applied interproximal		
Width 5 mm	3 m	591505
Width 6 mm	3 m	591506
Width 7 mm	3 m	591507

ROEKO Strip	600009
Transparent Matrix bands made from polyester for composite fillings	
Width 9 mm	15 m

ROEKO Steel Matrix Band (regular)		
Stainless steel, regular (0.03mm)		
Width 5 mm	3 m	590305
Width 6 mm	3 m	590306
Width 7 mm	3 m	590307

Stainless steel, regular (0.05mm)		
Width 5 mm	3 m	590505
Width 6 mm	3 m	590506
Width 7 mm	3 m	590507

ROEKO Steel separating strips



For approximal separation. Coated on one side (50 micron).

ROEKO Steel separating strips		
4 mm	12 pcs	570004
6 mm	12 pcs	570006
8 mm	12 pcs	570008

ROEKO Abrasive- and Polishing Strips



For smoothing of interproximal spaces not accessible or difficult to access with other instruments. Made of abrasion resistant polyester for final finishing of the filling.

ROEKO Abrasive- and Polishing Strips

30 micron, width 4 mm	10 m	560030
40 micron, width 4 mm	10 m	560040
60 micron, width 4 mm	10 m	560060

DIATECH SwissFlex Kit



For contouring, finishing and high luster polishing of composite, amalgam, glass ionomer cement, semi-precious metal and precious metal.

Colour	Indication	Description
Black	contouring	coarse (70µm)
Blue	finishing	medium (50µm)
Red	polishing	fine (30µm)
White	high lustre	ultra-fine (5µm)

DIATECH SwissFlex Kit				250095AA	
Content				Content	
3 x 702RA		232018AA			
25 x 703UM		232019AA	25 x 707UM		232023AA
25 x 704UM		232020AA	25 x 708UM		232024AA
25 x 705UM		232021AA	25 x 709UM		232025AA
25 x 706UM		232022AA	25 x 710UM		232026AA
20 x 700UM					232016AA
20 x 701UM					232017AA

DIATECH Finishing & Polishing Kits for Composite



Instruments selected by Dr. Paulo Monteiro for brilliant preparation, finishing and polishing of composite restorations.

DIATECH Brilliant Dentistry Kit				60020067	
Content				Content	
1 x G898-314-016-11-ML		200736AA	1 x G864-314-014-12-F		200572AA
1 x G368-314-023-5-ML		200015AA	1 x G853-314-008-3.5-XF		200408AA
1 x G368-314-023-5-F		200014AA	1 x C246-314-012-4.5-F		235049AA
1 x G801-314-023-F		200095AA			
1 x 2301RA		230291AA	1 x 24SG14RA		60019880
1 x 23SG14RA		60019878	1 x 7001RA		232002AA
1 x 2401RA		230299AA	1 x 7003RA		232006AA

Two-step diamond polishing system for smoothing, pre- and high luster polishing of composites.



DIATECH ShapeGuard Composite Polishing Plus Kit				60019916	
Content				Content	
1 x 2301RA		230291AA	1 x 2401RA		230299AA
1 x 2311RA		230295AA	1 x 2411RA		230303AA
1 x 23SG14RA		60019878	1 x 24SG14RA		60019880



High performance polishers for smoothing and high gloss polishing of all composite materials.

DIATECH Composite 1step Polishing Kit						250093AA
Content						
2 × 9101RA		230264AA	2 × 9111RA		230267AA	
2 × 9103RA		230265AA	2 × 9121RA		230271AA	

DIATECH Finishing & Polishing Kits for BRILLIANT Crios

This kit contains everything that is needed for chairside finishing and polishing of BRILLIANT Crios. The DIATECH polishers achieve an outstanding high luster polishing result and a tooth-like enamel gloss. Due to its flexible lamellas, the spiral DIATECH ShapeGuard adapts to any surface and provides uniform pressure distribution.



DIATECH Finishing & Polishing Kit for BRILLIANT Crios						60021732
Content			Content			
1 × G368-314-023-5-F		200014AA	1 × 24SG11RA		60019879	
1 × 2303RA		230293AA	1 × 24SG14RA		60019880	
1 × 23SG11RA		60019877	1 × 7001RA		232002AA	
1 × 23SG14RA		60019878				

This kit contains a range of burs and polishers that are needed for labside finishing and polishing of BRILLIANT Crios. The DIATECH polishers achieve an outstanding high luster polishing result.



DIATECH Lab Finishing & Polishing Kit for BRILLIANT Crios						60021819
Content			Content			
1 × 860-104-012-5-F		240142AA	1 × 9104HP		60021794	
1 × 2303RA		230293AA	1 × 9121HP		60021795	

S.P.E.C. 3



Performance Without Compromise

S.P.E.C. 3 meets the highest demands of performance in the dental practice. The stable, lightweight aluminium handpiece and the 360° rotating light guide provide convenient handling. Three different curing modes provide performance without compromise, enabling variable use to meet the individual requirements of dentists. A high performance, interchangeable lithium polymer battery provides over 300 10 s cures between recharges.

S.P.E.C. 3 LED Kit Euro		60013942		
S.P.E.C. 3 LED Euro Plug				
Light Shield	2 pcs		Light Shield 8 × 11 mm	20 pcs 60013948
Eye Shield	1 pc		Light Shield 11 × 11 mm	20 pcs 60014360
Barrier Sleeves	100 pcs		Barrier Sleeves	100 pcs 60013949
S.P.E.C. 3 Charger Base, Power Supply			Black Light Guide, 8 mm Turbo-Tip	60019326
			Black Light Guide, 11 mm	60019327
			Clear Light Guide, 8 mm Turbo-Tip	60013950
			Clear Light Guide, 11 mm	60013951
			Battery Module	60013952
			S.P.E.C. 3 Charger Base	60013953
			Power Supply	60013955
			Eye Shield	60014444

S.P.E.C. 3 LED Kit UK		60013943		
S.P.E.C. 3 LED UK Plug				
Light Shield	2 pcs			
Eye Shield	1 pc			
Barrier Sleeves	100 pcs			
S.P.E.C. 3 Charger Base, Power Supply				

Coltolux LED





The Coltolux LED combines a slim curing probe with high output in a cordless lightweight penstyle design. The Coltolux LED's heat sink technology reduces the amount of heat emission and eliminates the need for active cooling.

Coltolux LED Kit		C7970		
Handpiece				
Charger Base				
Power Supply				
Light Shields	3 pcs		Light Lenses, screw-on	25 pcs C7974
Light Lenses	5 pcs		Barrier Sleeves	1000 pcs C7957
Barrier Sleeves	100 pcs		Light Shields	100 pcs C7944
			Light Lenses, attachable	25 pcs C7972
			Light Shields, attachable	25 pcs C7990

Cool Temp NATURAL

Temporary crowns and bridge material based on bis-acryl composite, available in automix cartridges and automix syringes.

- 3 fluorescent shades
- low shrinkage
- excellent polishability
- low polymerisation temperature

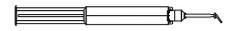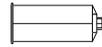

Cool Temp Starter Kit	5800
Cool Temp cartridge A2	85 g
Cool Temp cartridge A3.5	85 g
Mixing Tip Blue pointed	20 pcs
SYNERGY D6 Flow A2/B2	2.3 g
SYNERGY D6 Flow A3.5/B3	2.3 g
Bent Flow Application Needles	4 pcs

Cool Temp Refill Cartridge A1	5815
Cool Temp Cartridge A1	85 g
Mixing Tip Blue pointed	10 pcs

Cool Temp Refill Cartridge A2	5805
Cool Temp Cartridge A2	85 g
Mixing Tip Blue pointed	10 pcs

Cool Temp Refill Cartridge A3.5	5810
Cool Temp Cartridge A3.5	85 g
Mixing Tip Blue pointed	10 pcs

Cool Temp Automix Syringe 8.5 g Refill A1	5820
Cool Temp Automix Syringe 8.5 g A1	2 pcs
Mixing Tip Brown pointed	8 pcs

Cool Temp Automix Syringe 8.5 g Refill A2	5824
Cool Temp Automix Syringes 8.5 g A2	2 pcs
Mixing Tip Brown pointed	8 pcs



Cool Temp Automix Syringe 8.5 g Refill A3.5	5825
Cool Temp Automix Syringe 8.5 g A3.5	2 pcs
Mixing Tip Brown pointed	8 pcs

Accessories		
Dispenser	1 pc	6900
Mixing Tip Blue pointed	40 pcs	7774
Mixing Tip Brown pointed	40 pcs	6748
Separator Intraoral	5 ml	7235

DuoTEMP

Dual curing temporary filling material, eugenol-free and radiopaque for temporary fillings, temporary fillings for endodontic applications and temporary fillings for inlay- and onlay-preparations. Available in universal white.

- dual curing
- excellent marginal fit
- easy to remove
- one material for several indications

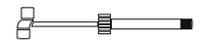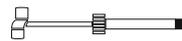

DuoTEMP Intro Kit	5830
Syringes	2 × 5 g
Separator Intraoral	5 ml

DuoTEMP Eco Pack	7090
Syringes	5 × 5 g


DuoTEMP Single Pack	5831
Syringe	5 g

Coltosol F



Temporary, eugenol-free filling material for temporary fillings and temporary root canal seals. Available in universal white.

- fluoride-releasing X-ray opaque
- easy adaption and adhesion in the cavity
- no mixing, easy to remove
- self-curing under the action of saliva

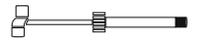

Coltosol F Single Pack	5906
Coltosol F in Jar	38 g

Coltosol F in Syringes	5930
Syringes	5 × 8 g

Coltosol F Eco Pack	5911
Coltosol F in Jar	3 × 38 g

Intraoral Separator



For intraoral use for easy removal of temporaries.





Intraoral Separator	7235
Bottle	5 ml

TempoSIL 2

Temporary PVS cement material for different surfaces and indications.

- secure adhesion of provisional restorations to natural tooth structure, core build up material and metal alloys as well as zirconia
- simple application as well as easy removal
- fast setting time
- two shades for esthetic applications

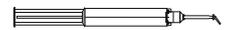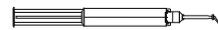

TempoSIL 2 Trial Pack	3686
TempoSIL 2 dentin	5 ml
TempoSIL 2 white	5 ml
Mixing Tips Brown long fine	15 pcs

TempoSIL 2 Dentin Refill	6728
TempoSIL 2 dentin	4 × 5 ml

TempoSIL 2 Dentin Intro Kit	6727
TempoSIL 2 dentin	2 × 5 ml
Mixing Tips Brown long fine	15 pcs

TempoSIL 2 White Refill	6730
TempoSIL 2 white	4 × 5 ml

TempoSIL 2 White Intro Kit	6729
TempoSIL 2 white	2 × 5 ml
Mixing Tips Brown long fine	15 pcs

Mixing Tips Brown long fine	6749
Mixing Tips	40 pcs

Composite Enamel Shells
Restoration
CAD/CAM Solutions

COMPONEER®

The direct composite veneering system for highly aesthetic anterior restorations.

COMPONEER simplifies build-up and surface design in the anterior region. At the same time, it allows minimally invasive preparation of the tooth substance and individual matching. The innovative micro-retentive surface increases wettability and strengthens sustainable bonding.

Available in up to 3 shades, sizes S-XL and shapes 14-24 and 43-33.



BRILLIANT Crios

Reinforced composite bloc for indirect CAD/CAM single tooth restorations.

BRILLIANT Crios is distinguished by superior grinding precision and outstanding mechanical properties. Together with a shock-absorbing effect and a natural bite feeling, BRILLIANT Crios is the ideal CAD/CAM material for permanent single tooth restorations.

Available in sizes 12 and 14 including 9 Low Translucent and 4 High Translucent shades.



COMPONEER®

Surprise your patients with a new smile – in only one session!

Innovative. Time-saving. Surprisingly easy. COMPONEER is the Direct Composite Veneering System used for quick, easy and save restorations of single or multiple teeth. This offers new perspectives for you and your patients. So both of you have a reason to smile.

www.COMPONEER.info



THE SMILE TO GO.



Die COMPONEER® benefits:

No laboratory required | One session | Naturally aesthetic corrections using freehand technique | Easy application with prefabricated composite veneers | Brilliant result | Attractive added value

www.coltene.com

 COLTENE

COMPONEER

COMPONEER - The Smile to go.

COMPONEER Direct Composite Veneers are polymerized prefabricated nano-hybrid composite enamel shells. COMPONEER eliminates laborious configuration of anatomical shape and surface, as well as time-consuming preparation by freehand technique. At the same time, COMPONEER can be adapted as with the individual situation anytime by using composite. The extremely thin veneer thicknesses of 0.3 mm enable preparation with minimal damage to tooth substance.

The polished and naturally designed surface gives the restoration its vital character. The micro-retentive inner surface increases wettability and strengthens sustainable bonding.

Whether incisal edge extension, covering tooth discolouration or malpositioning of teeth: with COMPONEER, aesthetic corrections are easy to achieve and finish in a single session.

Properties in clinical use.

1. Initial situation: discoloured teeth, extensive caries, malposition, incorrect axial alignment.



2. Select enamel and dentine shade with shade guide prior to isolation, ideally under daylight lamp.



3. The perfectly matching COMPONEER is selected using the bluish transparent Contour Guides.




4. Rubberdam adhesion technique facilitates functional working.



5. Prior to opening the sealed COMPONEER, the shape is verified again following preparation.



6. Careful grinding of the COMPONEER to give an exact fit.

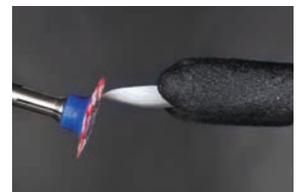

7. The composite is distributed on the tooth using the MBC modelling instrument. The placer is used for stress-free application of the COMPONEER to its final position.



8. Final check for perfect edges before removing the rubberdam.



COMPONEER SYSTEM KITS



1301

COMPONEER PREMIUM System Kit (84 COMPONEER)	Tips	Syringes
	1300	1301
each upper Set: 11, 12, 13, 21, 22, 23		
1 × UJ S	Enamel Universal	
1 × UJ M	Enamel Universal	
2 × UJ L	Enamel Universal	
1 × UJ XL	Enamel Universal	
1 × UJ S	Enamel White Opalescent	
1 × UJ M	Enamel White Opalescent	
2 × UJ L	Enamel White Opalescent	
1 × UJ XL	Enamel White Opalescent	
each lower Set: 31, 32, 33, 41, 42, 43		
1 × LJ S	Enamel Universal	
1 × LJ M	Enamel Universal	
1 × LJ S	Enamel White Opalescent	
1 × LJ M	Enamel White Opalescent	
Material and Accessories*		

* Material and Accessories
COMPONEER Contour Guide, (upper: S, M, L, XL; lower: S, M)
One Coat Bond (1 × 1.2 ml), Etchant Gel S (1 × 2.5 ml)
SYNERGY D6 Dentin A1/B1, A2/B2; SYNERGY D6 Enamel White Opalescent, Enamel Universal SYNERGY D6 Flow Enamel Universal (1 × 2.3 g Syringe) (each colour: 10 × 0,25 g Tips; 1 × 4 g Syringe)
1 Shade Guide Componeer-SYNERGY D6
15 Bent Appl. Needles "Flow"
1 MBC Modelling Instrument
1 Holder and 12 Holder Caps
1 Placer, 2 Placer Adapter, 10 Placer Stamps
1 Brushholder black and 50 Brushes
1 Comprepol Plus Cup and Point, Composhine Plus Cup and Point (1 pc each)
1 SwissFlex Trial Kit (1 Mandrel, 11 Discs)



1305

COMPONEER BASIC System Kit (36 COMPONEER)	Tips	Syringes
	1304	1305
each upper Set: 11, 12, 13, 21, 22, 23		
1 × UJ M	Enamel Universal	
1 × UJ L	Enamel Universal	
1 × UJ M	Enamel White Opalescent	
1 × UJ L	Enamel White Opalescent	
each lower Set: 31, 32, 33, 41, 42, 43		
1 × LJ M	Enamel Universal	
1 × LJ M	Enamel White Opalescent	
Material and Accessories*		



60014222


COMPONEER INTRO System Kit (24 COMPONEER)	Tips	Syringes
	60014221	60014222
each upper Set: 11, 12, 13, 21, 22, 23		
1 × UJ M	Enamel Universal	
1 × UJ L	Enamel Universal	
1 × UJ M	Enamel White Opalescent	
1 × UJ L	Enamel White Opalescent	
Material and Accessories*		

COMPONEER Refills UJ

A wide selection of sizes and shades

Choose from the wide range of maxillary veneers (maxilla) available in sizes S, M, L and XL, as well as mandibular veneers (mandible) in sizes S or M. Three natural shades allow maximum aesthetics for every indication.

Upper jaw						
FDI _i	13	12	11	21	22	23

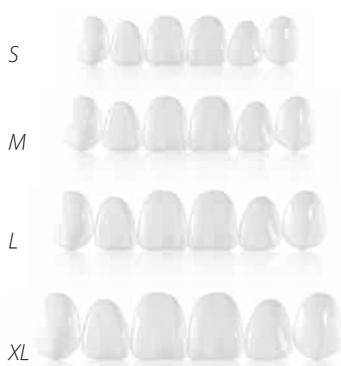

COMPONEER Sets UJ	Dentin Bleach Opaque	Enamel White Opalescent	Enamel Universal
COMPONEER Set UJ S , 6 Veneers: 11, 12, 13, 21, 22, 23		1311	1316
COMPONEER Set UJ M , 6 Veneers: 11, 12, 13, 21, 22, 23	60014500	1312	1317
COMPONEER Set UJ L , 6 Veneers: 11, 12, 13, 21, 22, 23	60014509	1313	1318
COMPONEER Set UJ XL , 6 Veneers: 11, 12, 13, 21, 22, 23	60014516	60014010	60014012
COMPONEER Set UJ Premolar S , 2 Veneers: 14/24	60014523	60014393	60014392
COMPONEER Set UJ Premolar L , 2 Veneers: 14/24	60014524	60014396	60014394

Upper jaw		
FDI _i	14	24



COMPONEER Refills UJ				
COMPONEER Refill UJ S , 2 Veneers	11		1321	1351
	12		1322	1352
	13		1323	1353
	21		1324	1354
	22		1325	1355
	23		1326	1356
COMPONEER Refill UJ M , 2 Veneers	11	60014502	1327	1357
	12	60014504	1328	1358
	13	60014505	1329	1359
	21	60014506	1330	1360
	22	60014507	1331	1361
	23	60014508	1332	1362
COMPONEER Refill UJ L , 2 Veneers	11	60014510	1333	1363
	12	60014511	1334	1364
	13	60014512	1335	1365
	21	60014513	1336	1366
	22	60014514	1337	1367
	23	60014515	1338	1368
COMPONEER Refill UJ XL , 2 Veneers	11	60014517	60013998	60014004
	12	60014518	60013999	60014005
	13	60014519	60014000	60014006
	21	60014520	60014001	60014007
	22	60014521	60014002	60014008
	23	60014522	60014003	60014009

COMPONEER Refills LJ

Lower jaw						
FDI	43	42	41	31	32	33




COMPONEER Sets LJ	Dentin Bleach Opaque	Enamel White Opalescent	Enamel Universal
COMPONEER Set LJ S , 6 Veneers: 31, 32, 33, 41, 42, 43		1314	1319
COMPONEER Set LJ M , 6 Veneers: 31, 32, 33, 41, 42, 43		1315	1320

COMPONEER Refills LJ			
COMPONEER Refill LJ S , 2 Veneers	31	1339	1369
	32	1340	1370
	33	1341	1371
	41	1342	1372
	42	1343	1373
	43	1344	1374
	COMPONEER Refill LJ M , 2 Veneers	31	1345
32		1346	1376
33		1347	1377
41		1348	1378
42		1349	1379
43		1350	1380



Order a COMPONEER Kit and receive your patient info kit free of charge. The order card is in your COMPONEER kit.

COMPONEER Info Kit D	60013798
DVD and CD, Patient brochure, Poster	

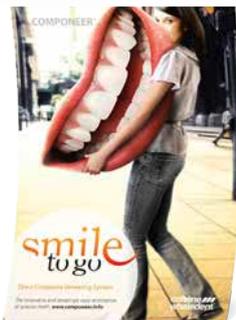

COMPONEER Accessories



Additional material for a perfect outcome

Instruments			
MBC Modelling Instrument	1 pc	60011110	
Holder	1 pc	60011105	
Holder Caps	40 pcs	60011106	
Placer	1 pc	60011107	
Placer Adapter	5 pcs	60011108	
Placer Stamps	50 pcs	60011109	
Brushholder black	1 pc	60011111	
Brushes	500 pcs	60011112	

Material	
Etchant Gel S	see page 10
One Coat Bond	see page 11
SYNERGY D6	see page 19
SYNERGY D6 Flow	see page 20

Preparation, Finishing, Polishing	
Interproximal Polishing	
DIATECH SwissFlex Kit	see pag. 25 250095AA

Contour Guides	
UJ Small	7000
UJ Medium	7001
UJ Large	7002
UJ X-Large	60014011
UJ Premolar	60014401
LJ Small	7003
LJ Medium	7004

Shade Guides		
Shade Guide SYNERGY D6	1 pc	8410
Shade Guide COMPONEER, SYNERGY D6 BL/O	1 pc	60013812



The selected, coarse, blue-coded diamonds are ideally suited for prep-work, the finer red and white coded diamonds for finishing and contouring. The specially designed finest diamond (orange) and the polishers (purple for prepolishing, blue for high luster polishing) improve the quality of margins and surface gloss. The length and design of the instruments enables cervical work at an optimal angle.

COMPONEER Preparation and Finishing Kit by Dr. Mario Besek				60013844	
Preparation, Finishing, Polishing					
1 × G898-314-016-11-ML		200736AA	1 × G860-314-012-10-UF		60013866
1 × G368-314-023-5-ML		200015AA	1 × 2311RA		230295AA
1 × G368-314-023-5-F		200014AA	1 × 2303RA		230293AA
1 × G864-314-014-12-F		200572AA	1 × 2411RA		230303AA
1 × G801-314-023-F		200095AA	1 × 2403RA		230301AA
1 × G863-314-012-10-UF		200561AA	1 × 7003RA		232006AA

Ceramics – a smart solution in the kitchen.
Composite – a BRILLIANT solution in the practice.



BRILLIANT Crios

Reinforced composite bloc for permanent restoration

- High flexural strength – resistant restorations
- Tooth-like modulus of elasticity – shock absorbing
- Wear resistant and gentle to the antagonist



www.coltene.com



COLTENE

BRILLIANT Crios



BRILLIANT Crios is a reinforced composite bloc. The material combines all the advantages of an innovative submicron hybrid composite material with those of a CAD/CAM fabrication process for reliable, aesthetic and fast restorations without a separate firing process. Consistently controlled fabrication with constant thermal curing, gives the reinforced composite blocs excellent mechanical properties. Two sizes and two translucencies with 13 shades offer a broad range for a daily use in the dental practice.

- High flexural strength for resistant restorations
- Tooth-like modulus of elasticity for a shock-absorbing effect and pleasant bite feeling
- High resistance and low abrasion on antagonist
- Exceptional grinding precision for more freedom of preparation
- Can be modified and repaired



BRILLIANT Crios Intro Kit CEREC 12	60020095
A2 LT 12	1 pc
A3 LT 12	1 pc
A3.5 LT 12	1 pc
A2 HT 12	1 pc
A3 HT 12	1 pc
ONE COAT 7 UNIVERSAL Single Dose	5 x 0.1 ml

BRILLIANT Crios Intro Kit CEREC 14	60020097
A2 LT 14	1 pc
A3 LT 14	1 pc
A3.5 LT 14	1 pc
A2 HT 14	1 pc
A3 HT 14	1 pc
ONE COAT 7 UNIVERSAL Single Dose	5 x 0.1 ml



BRILLIANT Crios Refills, 5 blocs – CEREC		
	Size 12	Size 14
Low Translucent		
Bleach	60019998	60019985
A1	60019999	60019986
A2	60020000	60019987
A3	60020001	60019988
A3.5	60020002	60019989
B1	60020003	60019990
B2	60020004	60019991
B3	60020005	60019992
C2	60020006	60019993
High Translucent		
A1	60020007	60019994
A2	60020008	60019995
A3	60020009	60019996
B1	60020010	60019997

Core Build-up
Cements
Restoration
Parapulpal pins




	SoloCem®	DuoCem®	ParaCore®
Description	Self-adhesive, dual-curing resin cement	Dual-curing resin cement	Dual-curing core & resin cement
Adhesive system	No adhesive necessary	ParaBond A.R.T. Bond One Coat Bond ONE COAT 7 UNIVERSAL	ParaBond
Shades	Dentine White opaque Translucent	Dentine White opaque Translucent	Dentine White opaque Translucent
INDICATIONS			
Crowns & bridges	+++	+++	++
Inlay / Onlay	+++	+++	++
Posts	+++	+++	+++
Implant abutments	++	+	
Core build-up			+++

+ possible
 ++ recommended
 +++ highly recommended

ParaCore 5 ml



Dual curing core material and resin cement in 5 ml automix-syringe.

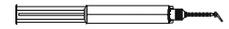

ParaCore 5ml Intro Kit	5885
ParaCore Dentin	1 × 5 ml
ParaCore White	1 × 5 ml
ParaCore Dentin SLOW	1 × 5 ml
ParaCore Trans SLOW	1 × 5 ml
ParaBond Non-Rinse Conditioner + Adhesive A & B	3 × 3 ml
Mixing Tip Short Super Fine	20 pcs

ParaCore SLOW 5ml Intro Kit	60013753
ParaCore Dentin SLOW	2 × 5 ml
ParaCore Trans SLOW	2 × 5 ml
ParaBond Non-Rinse Conditioner + Adhesive A & B	3 × 3 ml
Mixing Tip Short Super Fine	20 pcs

ParaCore 5ml White Refill	5886
ParaCore White	2 × 5 ml
Mixing Tip Short Super Fine	20 pcs

ParaCore 5ml Dentin Refill	5887
ParaCore Dentin	2 × 5 ml
Mixing Tip Short Super Fine	20 pcs each

ParaCore 5ml Dentin SLOW Refill	60011391
ParaCore Dentin SLOW	2 × 5 ml
Mixing Tip Short Super Fine	20 pcs

ParaCore 5ml Trans SLOW Refill	60011392
ParaCore Trans SLOW	2 × 5 ml
Mixing Tip Short Super Fine	20 pcs

Mixing Tip Short Super Fine	6747
Mixing Tip, Ø 1 mm	40 pcs

Mixing Tip Short Fine	6759
Mixing Tip, Ø 1,8 mm	40 pcs

ParaCore 25 ml



Dual curing core build-up material in 25 ml cartridge.



ParaCore 25ml Dentin Intro Kit	1500
ParaCore Dentin	1 × 25 ml
COLTENE Dispenser 25ml	1 pc
ParaBond Non-Rinse Conditioner + Adhesive A & B	3 × 3 ml
Mixing Tip Yellow / Oral Tip Yellow	20 pcs each

ParaCore 25ml Dentin Kit	1501
ParaCore Dentin	1 × 25 ml
ParaBond Non-Rinse Conditioner + Adhesive A & B	3 × 3 ml
Mixing Tip Yellow / Oral Tip Yellow	20 pcs each

ParaCore 25ml White Intro Kit	1502
ParaCore White	1 × 25 ml
COLTENE Dispenser 25ml	1 pc
ParaBond Non-Rinse Conditioner + Adhesive A & B	3 × 3 ml
Mixing Tip Yellow / Oral Tip Yellow	20 pcs each

ParaCore 25ml White Kit	1503
ParaCore White	1 × 25 ml
ParaBond Non-Rinse Conditioner + Adhesive A & B	3 × 3 ml
Mixing Tip Yellow / Oral Tip Yellow	20 pcs each

ParaCore 25ml Dentin Refill	5853
ParaCore Dentin	1 × 25 ml
Mixing Tip Yellow / Oral Tip Yellow	20 pcs each

ParaCore 25ml White Refill	5854
ParaCore White	1 × 25 ml
Mixing Tip Yellow / Oral Tip Yellow	20 pcs

Mixing Tips Yellow	6550
Mixing Tip Yellow	40 pcs

Oral Tips Yellow	6555
Oral Tip Yellow	100 pcs

COLTENE Dispenser 25 ml	4470
-------------------------	------

ParaBond Adhesive	7486
ParaBond Non-Rinse Conditioner	1 × 3 ml
ParaBond Adhesive A	1 × 3 ml
ParaBond Adhesive B	1 × 3 ml

ParaBond Adhesive A & B Refill	7493
ParaBond Adhesive A	1 × 3 ml
ParaBond Adhesive B	1 × 3 ml

ParaBond Non-Rinse Conditioner Refill	7494
ParaBond Non-Rinse Conditioner	1 × 3 ml

DuoCem



Dual curing resin cement.

COLTENE cements have been proving their worth for many years, convincing dentists the world over. DuoCem thus builds on the strengths of these recognised brands, combining them with the benefits of an automix syringe. This combination results in an all-purpose resin cement that offers excellent technical characteristics and can be applied with ease. Thus making DuoCem the ideal cement for routine bonding tasks in the dental practice.



DuoCem Intro Kit	60014381
DuoCem Dentin	1 × 5 ml
DuoCem Trans	1 × 5 ml
Mixing Tip Short Fine	10 pcs
Mixing Tip Short Super Fine	10 pcs

DuoCem Refill Dentin	60014382
DuoCem Dentin	1 × 5 ml
Mixing Tip Short Fine	5 pcs
Mixing Tip Short Super Fine	5 pcs

DuoCem Refill Trans	60020092
DuoCem Trans	1 × 5 ml
Mixing Tip Short Fine	5 pcs
Mixing Tip Short Super Fine	5 pcs

DuoCem Refill White Opaque	60014383
DuoCem White Opaque	1 × 5 ml
Mixing Tip Short Fine	5 pcs
Mixing Tip Short Super Fine	5 pcs

Mixing Tip Short Super Fine	6747
Mixing Tip, Ø 1 mm	40 pcs

Mixing Tip Short Fine	6759
Mixing Tip, Ø 1,8 mm	40 pcs

SELF-ADHESIVE DUAL-CURING RESIN CEMENT



SoloCem[®]

One step reliable cementation

- easy to use and time saving
- high bonding strengths without additional adhesive
- low shrinkage
- antibacterial zinc oxide
- high radiopacity



Intro Kit
REF 60013986

SoloCem

Self-adhesive dual curing resin cement.

Secure cementing does not depend on the number of working steps. Self-adhesive SoloCem achieves reliable values – with no additional bonding. This is the result of monomers contained and the composite-like formulation. The convenient automix syringe and the ready-to-use mixing tips offer a simple and time-saving application of SoloCem.

SoloCem Intro Kit		60013986
SoloCem Dentin	1 × 5 ml	
SoloCem Trans	1 × 5 ml	
Mixing Tip Short Fine	10 pcs	
Mixing Tip Short Super Fine	10 pcs	

SoloCem Refill Dentin		60014062
SoloCem Dentin	1 × 5 ml	
Mixing Tip Short Fine	5 pcs	
Mixing Tip Short Super Fine	5 pcs	

SoloCem Refill Trans		60019840
SoloCem Trans	1 × 5 ml	
Mixing Tip Short Fine	5 pcs	
Mixing Tip Short Super Fine	5 pcs	

SoloCem Refill White Opaque		60014061
SoloCem White Opaque	1 × 5 ml	
Mixing Tip Short Fine	5 pcs	
Mixing Tip Short Super Fine	5 pcs	



Mixing Tip Short Super Fine		6747
Mixing Tip, Ø 1 mm		40 pcs

Mixing Tip Short Fine		6759
Mixing Tip, Ø 1,8 mm		40 pcs

MAX Pin



Self-threading parapulpal retention pins with shoulder stop and rounded head design for good adaptation. Corrosion-free, titanium alloy. Self-alignment during placement.

Complete Kit 017, blue	M61
Diameter: .017" / .425 mm	
MAX Pin	25 pcs
MAX Drill	2 pcs
Universal hand driver	1 pc

Complete Kit 021, purple	M71
Diameter: .021"/.525 mm	
MAX Pin	25 pcs
MAX Drill	2 pcs
Universal hand driver	1 pc

Bulk Kit 021, purple	M73
Diameter: .021"/.525 mm	
MAX Pin	60 pcs

Bulk Kit 017, blue	M63
Diameter: .017"/.425 mm	
MAX Pin	60 pcs

Accessories	
MAX Drills (.017" × .075"/.425 mm × 1.9 mm) blue (depth limiting)	M91
MAX Drills (.021" × .095"/.525 mm × 2.4 mm) purple (depth limiting)	M92
TMS Regular and Minikin hand wrench (Complete/Two-in-One/Self-Shearing)	B59-3
TMS Minim and Minim 2/1 hand wrench	B59-5
TMS Minuta hand wrench	B59-10
Universal hand driver	L590

TMS LINK / TMS LINK Plus




Self-threading parapulpal retention pins. Alpha-beta titanium alloy or gold plated stainless steel. Shoulder stop and smooth self-alignment during placement. Available as single shear pins and double shear pins. The system is completed with the colour coded KODEX®-drills to ensure an optimal preparation of the canal.

Titanium Alloy


TMS LINK Minikin, red	L811
Single-Shear Pins, Diameter: .017"/.425 mm	
Single-Shear Pins	20 pcs
Kodex drill K91	1 pc
Universal hand driver	1 pc

TMS LINK Minikin, red	L812
Diameter: .017"/.425 mm (no drills)	
Bulk Kit Single-Shear Pins	50 pcs

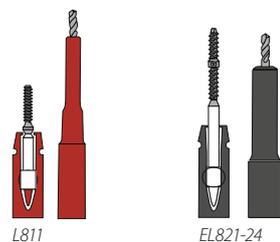

TMS LINK Plus Minim, silver	EL821-24
Single-Shear Pins Diameter: .021"/.525 mm	
Single-Shear Pins	24 pcs
Kodex drill K92	1 pc
Universal hand driver	1 pc



TMS LINK Plus Minim, silver	EL822-60
Diameter: .021"/.525 mm (no drills)	
Single-Shear Pins	60 pcs

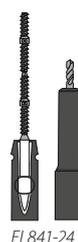

TMS LINK Plus Minim 2/1, silver	EL841-24
Double-Shear Pins Diameter: .021"/.525 mm	
Double-Shear Pins	24 pcs
Kodex drill K92	2 pcs
Universal hand driver	1 pc

TMS LINK Plus Minim 2/1 Bulk, silver	EL842-60
Diameter: .021"/.525 mm (no drills)	
Double-Shear Pins	60 pcs

TMS LINK

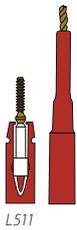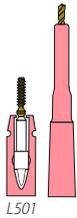

Gold-Plated Stainless Steel

TMS LINK Minuta, pink	L501
Diameter: .0135"/.350 mm	
Single-Shear Pins	20 pcs
Kodex drill K89	1 pc
Universal hand driver	1 pc

TMS LINK Minikin, red	L511
Diameter: .017"/.425 mm	
Single-Shear Pins	20 pcs
Kodex drill K91	1 pc
Universal hand driver	1 pc

TMS LINK Minim, silver	L521
Diameter: .021"/.525 mm	
Single-Shear Pins	20 pcs
Kodex drill K92	1 pc
Universal hand driver	1 pc

TMS LINK Minim 2/1, silver	L531
Diameter: .021"/.525 mm	
Double-Shear Pins	20 pcs
Kodex drill K92	2 pcs
Universal hand driver	1 pc

TMS LINK Regular, gold	L541
Diameter: .027"/.675 mm	
Single-Shear Pins	20 pcs
Kodex drill K96	1 pc
Universal hand driver	1 pc

TMS LINK Regular 2/1, gold	L551
Diameter: .027"/.675 mm	
Double-Shear Pins	20 pcs
Kodex drill K96	2 pcs
Universal hand driver	1 pc

TMS LINK Minuta Bulk, pink	L502
Diameter: .0135"/.350 mm (no drills)	
Single-Shear Pins	50 pcs

TMS LINK Minikin Bulk, red	L512
Diameter: .017"/.525 mm (no drills)	
Single-Shear Pins	50 pcs


TMS LINK Minim Bulk, silver	L522
Diameter: .021"/.525 mm (no drills)	
Single-Shear Pins	50 pcs

TMS LINK Minim 2/1 Bulk, silver	L532
Diameter: .021"/.525 mm (no drills)	
Double-Shear Pins	50 pcs

TMS LINK Regular Bulk, gold	L542
Diameter: .027"/.675 mm (no drills)	
Single-Shear Pins	50 pcs

TMS LINK Regular 2/1 Bulk, gold	L552
Diameter: .027"/.675 mm (no drills)	
Double-Shear Pins	50 pcs


TMS LINK Plus

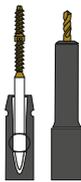


EL721-24



EL731-24

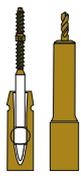

EL741-24

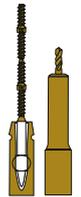

EL751-24

TMS LINK Plus Minim, silver	EL721-24
Diameter: .021"/.525 mm	
Single-Shear Pins	24 pcs
Kodex drill K92	1 pc
Universal hand driver	1 pc

TMS LINK Plus Minim 2/1, silver	EL731-24
Diameter: .021"/.525 mm	
Double-Shear Pins	24 pcs
Kodex drill K92	2 pcs
Universal hand driver	1 pc

TMS LINK Plus Regular, gold	EL741-24
Diameter: .027"/.675 mm	
Single-Shear Pins	24 pcs
Kodex drill K96	1 pc
Universal hand driver	1 pc

TMS LINK Plus Regular 2/1, gold	EL751-24
Diameter: .027"/.675 mm	
Double-Shear Pins	24 pcs
Kodex drill K96	2 pcs
Universal hand driver	1 pc

TMS LINK Plus Minim Bulk, silver	EL722-60
Diameter: .021"/.525 mm (no drills)	
Single-Shear Pins	60 pcs

TMS LINK Plus Minim 2/1 Bulk, silver	EL732-60
Diameter: .021"/.525 mm (no drills)	
Double-Shear Pins	60 pcs

TMS LINK Plus Regular Bulk, gold	EL742-60
Diameter: .027"/.675 mm (no drills)	
Single-Shear Pins	60 pcs

TMS LINK Plus Regular 2/1 Bulk, gold	EL752-60
Diameter: .027"/.675 mm (no drills)	
Double-Shear Pins	60 pcs

KODEX-Drills



Minuta pink	K89
.0135" × .051"/.350 × 1.3 mm depth-limiting	
	6 pcs

Minikin red	K90
.017" × .059"/.425 × 1.5 mm short shank, depth-limiting	
	6 pcs

Minim silver	K92
.021" × .079"/.525 × 2 mm depth-limiting	
	6 pcs

Regular gold	K96
.027" × .079"/.675 × 2 mm depth-limiting	
	6 pcs

Minikin red	K91
.017" × .059"/.425 × 1.5 mm long shank, depth-limiting	
	6 pcs

Minim silver	K93
.021" × .157"/.525 × 4 mm without depth-limitation	
	6 pcs

Regular gold	K97
.027" × .197"/.675 × 5 mm without depth-limitation	
	6 pcs

Universal hand driver	L590
	1 pc

TMS



TMS Regular Complete Kit T01	
Diameter (.027"/.675 mm)	
Self-Threading Pins	20 pcs
Kodex Drill K97	1 pc
Hand Wrench	1 pc

TMS Regular Bulk Kit T03	
Diameter (.027"/.675 mm)	
Self-Threading Pins	100 pcs

TMS Regular Refill Kit T02	
Diameter (.027"/.675 mm)	
Self-Threading Pins	50 pcs



TMS Minim Complete Kit T04	
Diameter (.021"/.525 mm)	
Self-Threading Pins	20 pcs
Kodex Drill K93	1 pc
Hand Wrench	1 pc

TMS Minim Bulk Kit T06	
Diameter (.021"/.525 mm)	
Self-Threading Pins	100 pcs

TMS Minim Refill Kit T05	
Diameter (.021"/.525 mm)	
Self-Threading Pins	50 pcs



TMS Regular Two-in-One Kit T07	
Diameter (.027"/.675 mm)	
Self-Threading Pins	2 x 20 pcs
Kodex Drill K96	1 pc
Hand Wrench	1 pc

TMS Regular Two-in-One Bulk Kit T09	
Diameter (.027"/.675 mm)	
Self-Threading Pins	2 x 100 pcs

TMS Regular Two-in-One Refill Kit T08	
Diameter (.027"/.675 mm)	
Self-Threading Pins	2 x 50 pcs



TMS Minim Two-in-One Kit T10	
Diameter (.021"/.525 mm)	
Self-Threading Pins	2 x 20 pcs
Kodex Drill K92	1 pc
Hand Wrench	1 pc

TMS Minim Two-in-One Bulk Kit T12	
Diameter (.021"/.525 mm)	
Self-Threading Pins	2 x 100 pcs

TMS Minim Two-in-One Refill Kit T11	
Diameter (.021"/.525 mm)	
Self-Threading Pins	2 x 50 pcs

TMS



TMS Regular Self-Shearing Complete Kit		T13
Diameter (.027"/.675 mm)		
Self-Threading Pins	20 pcs	
Kodex Drill K97	1 pc	
Hand Wrench	1 pc	

TMS Regular Self-Shearing Refill Kit		T14
Diameter (.027"/.675 mm)		
Self-Threading Pins	50 pcs	

TMS Regular Self-Shearing Bulk Kit		T15
Diameter (.027"/.675 mm)		
Self-Threading Pins	100 pcs	



TMS Minikin Complete Kit		T107
Diameter (.017"/.425 mm)		
Self-Threading Pins	20 pcs	
Kodex Drill K91	1 pc	
Hand Wrench	1 pc	

TMS Minikin Refill Kit		T108
Diameter (.017"/.425 mm)		
Self-Threading Pins	50 pcs	

TMS Minikin Bulk Kit		T109
Diameter (.017"/.425 mm)		
Self-Threading Pins	100 pcs	



TMS Minuta Complete Kit		T111
Diameter (.0135"/.350 mm)		
Self-Threading Pins	20 pcs	
Kodex Drill K89	1 pc	
Hand Wrench	1 pc	

TMS Minuta Refill Kit		T112
Diameter (.0135"/.350 mm)		
Self-Threading Pins	50 pcs	



Locations

CH **Coltène/Whaledent AG**

Feldwiesenstrasse 20
9450 Altstätten / Switzerland
T +41 71 757 53 00
F +41 71 757 53 01
info.ch@coltene.com

CN **Coltène/Whaledent China Trading Co. Ltd.**

Rm. 1103/1104, Bldg. B2, Wu Dong Plaza
#9 Chegongzhuang Street, West District
Beijing 100044 / P.R. China
T +86 10 6212 3251
F +86 10 6212 3252
info.cn@coltene.com

IT **Coltène Italia S.r.l.**

c/o Polo Tecnologico della Brianza
Via Alessandro Volta, 94 - Cond. Bianchi
20832 Desio (MB) / Italy
T +39 0362 62 67 32
F +39 0362 63 25 80
info.it@coltene.com

DE **Coltène/Whaledent GmbH + Co. KG**

Raiffeisenstraße 30
89129 Langenau / Germany
T +49 7345 805 0
F +49 7345 805 201
info.de@coltene.com

FR **Coltène/Whaledent S.a.r.l.**

Synergie Park
3 avenue Louis Néel
59260 Lezennes / France
T +33 320 64 64 00
F +33 320 64 64 03
info.fr@coltene.com

UK **Coltène/Whaledent Ltd.**

The President Suite
Kendal House, Burgess Hill
Victoria Way
West Sussex, RH15 9NF / UK
T +44 1444 235486
F +44 1444 870640
info.uk@coltene.com

US **Coltène/Whaledent Inc.**

235 Ascot Parkway
Cuyahoga Falls, Ohio 44223 / USA
T +1 330 916 8800
F +1 330 916 7077
info.us@coltene.com

IN **Coltène/Whaledent Private Ltd.**

106/107, Hallmark, Vasant Oscar
L.B.S Marg, Mulund West
Mumbai 400080, Maharashtra / India
T +91 22 25923626
F +91 22 25623165
info.in@coltene.com

BR **Vigodent SA Indústria e Comércio**







Rua Pesqueira 26 Bonsucesso
Rio de Janeiro CEP 21041-150 / Brazil
T +5521 3865 5600
F +5521 2280 8411
sac@vigodent.com.br

ES **Coltène Iberia S.L.U.**

PT c/Doctor Fleming, 3 - Bajo A
28220 Majadahonda, Madrid / Spain
T +34 91 63 86 492
F +34 91 63 86 279
info.ib@coltene.com

JP **Coltène Japan LLC**

2-2-28 2nd Floor, Nishikicho
190-0022 Tachikawa Tokyo / Japan
T +81 42 595 6945
F +81 42 595 6946
info.jp@coltene.com

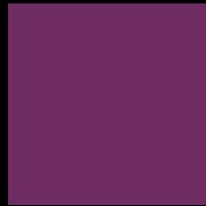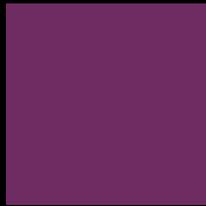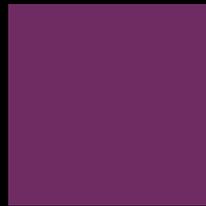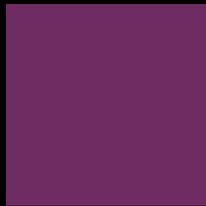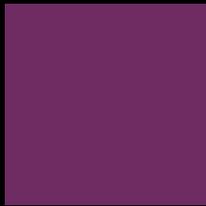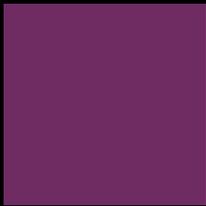

© 2017 Coltène/Whaledent AG – www.coltene.com



 COLTENE